

healthPAL™ Monitoring Glucose

Using HealthPAL with **TRUEbalance™** **TRUEresult™**

ONETOUCH Ultra **Abbott FreeStyle FREEDOM-Lite** **Abbott FreeStyle Lite** **BAYER Contour™**

1. Ensure that the glucose meter and HealthPAL are securely connected using the appropriate Smart Cable (See “Using Glucose Meters with MedApps Smart Cable”)
2. Once the devices are connected, the glucose meter will go into communication mode displaying either PC and/or a series of dashes depending on the glucose meter being used.
3. After a few seconds HealthPAL will receive the reading(s), then display and audibly announce the most current glucose reading. (“Your blood glucose reading is XXX.”)
4. The measurement(s) will be stored in HealthPAL’s memory and transmitted to server automatically when HealthPAL is inactive for 3 minutes.



Always follow manufacturer instructions for operation and safety for any accessory health monitors used with HealthPAL.

